


## Using the Pipette

- 1 Each pipette is single use. Unwrap the pipette from the top. Only touch the upper bulb of the pipette and never set the open pipette down on your work surface or touch the tip. Immediately dispose of the pipette after use.

- 2 To Use the pipette:
  - a. Squeeze the upper bulb
  - b. Submerge the pipette tip to the bottom of the sample tube
  - c. Release the upper bulb
  - d. Pause to verify that the pipette is:
    - Full of liquid
    - Free of bubbles or air gaps
    - Overflow fluid has entered the lower bulb
  - e. Dispense the sample into the sample port of the Visby Test by squeezing the upper bulb. Bubbles the sample port of the test are okay. Some overflow fluid will remain in the lower bulb

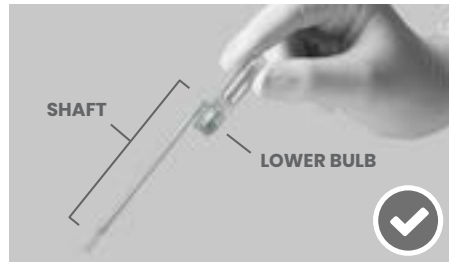

**Tip:** If the pipette has bubbles or an air gap, dispense the sample back into the sample tube and try aspirating again by slowly releasing the upper bulb.

- 3 When dispensing, some overflow fluid will remain in the lower bulb. Add only 1 pipette full of sample into the Visby Test. Do not add more or less volume.



- 4 Only use the Visby provided pipettes to load the Visby Device. Additional pipettes are provided if needed.

