


We're Implementing a New Sexual Health Test

~28 minute Chlamydia, Gonorrhea, Trichomoniasis
PCR results at the point of care

Test, talk and treat – in the same visit™

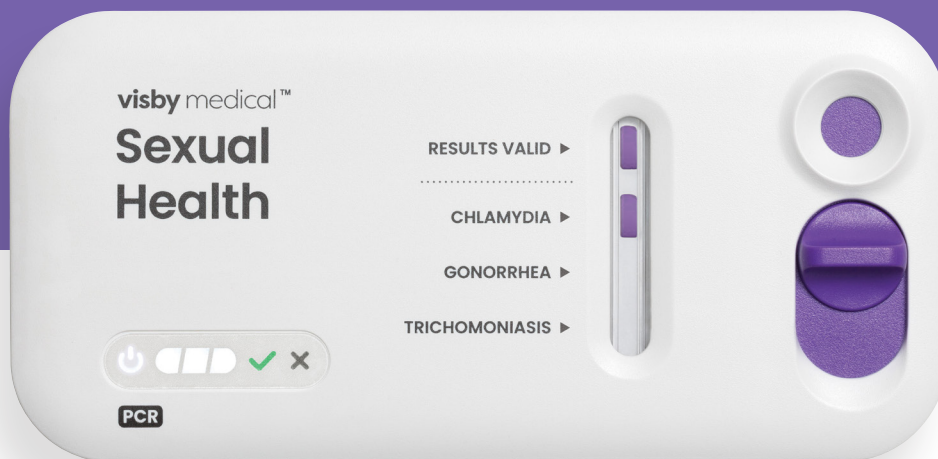

We have selected the CLIA-waived Visby Medical Sexual Health Test for rapid PCR testing at the point of care. We'll be able to test for three common STIs, get rapid results to discuss with patients, and prescribe appropriate treatment, all in the same visit.

- ✓ Reduce patient loss to follow-up
- ✓ Eliminate call backs
- ✓ Make treatment decisions at the point of care
- ✓ Reduce lab send out delays
- ✓ Avoid unnecessary antibiotics
- ✓ Improve the patient experience
- ✓ Streamline workflow
- ✓ Vaginal self-collected swab

✓ **RIGHT.**
📍 **HERE.**
⌚ **AND NOW.**

The World's Easiest PCR Test




1. Patient Self-Collects Sample

- ✓ No complicated cartridge assembly
- ✓ No special sample prep/extraction



2. Mix Sample

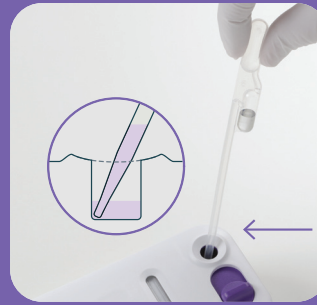

3. Load Sample



4. Flip the Switch

- ✓ No separate reader or instrument
- ✓ No maintenance

Users should read the complete test procedure before performing the test. Please refer to the quick reference guide and instructions for use for more information.

Specifications

- Time to Result: ~28 Minutes
- Sample type: Vaginal self-collected swab
- PPA: 97.4% (CT), 97.8% (NG), Sensitivity 99.3% (TV)
- NPA: 97.8% (CT), 99.1% (NG), Specificity 96.7% (TV)
- Storage: 36°F–86°F (2°C–30°C)
- Swab Stability in Visby Collection Media: 2°–30°C: 4 Hours, <–15°C: 90 days
- Single Use Pipettes included
- FDA Cleared & CLIA Waived

Visit www.visby.com/start to begin your journey with Visby.

You'll get helpful instructions to prepare to go live, tools to train staff, and links to download important documents.

Scan to access
on-demand
training
resources



Need further help?

Training & Implementation Support: visby.training@visby.com

Customer Support

Hours: Mon–Fri 8:00 am – 4:00 pm PST | Email: support@visby.com | Phone: 1-833-GoVisby (1-833-468-4729)

visby medical™

©2025 Visby Medical, Inc. All rights reserved. Visby Medical and the Visby Medical logo are registered trademarks of Visby Medical, Inc.

PL-000329 Rev A